

i fr it de es pl pt hu tr ru nl

- (fr) CE DISPOSITIF EST À USAGE UNIQUE**  
Contenu : 1 éponge contenant 215 ml de détergent enzymatique
- (it) ESCLUSIVAMENTE MONOUSO**  
Contenuto: 1 spugna con 215 ml di detergente enzimatico
- (de) NUR ZUM EINMALGEBRAUCH**  
Inhalt: 215 ml enzymatischem Reinigungsmittel
- (es) PARA UN SOLO USO**  
Contenido: 1 esponja con 215 ml de detergente enzimático
- (pl) WYŁĄCZNIE DO JEDNORAZOWEGO UŻYTKU**  
Zawartość: 1 gąbka nasączona 215 ml detergentu enzymatycznego
- (pt) APENAS PARA UTILIZAÇÃO ÚNICA**  
Conteúdo: 1 Esponja com 215 ml de Detergente Enzimático
- (hu) KIZÁRÓLAG EGYSZERI HASZNÁLATRA**  
Tartalom: 1 szivacs 215 ml enzimátikus mosószerrel
- (tr) TEK KULLANIMLİKTİR**  
İçindekiler: 215 ml Enzimatik Deterjan içeren 1 Sünger
- (ru) ТОЛЬКО ДЛЯ ОДНОРАЗОВОГО ПРИМЕНЕНИЯ**  
Содержимое: 1 губка с 215 мл ферментативного моющего средства
- (nl) UITSLUITEND VOOR EENMALIG GEBRUIK**  
Inhoud: 1 spons met 215 ml van enzymatisch wasmiddel



**REF 2D91QX**

**Revital-Ox™**  
Bedside Complete Pre-Cleaning Kit

**STERIS Corporation**  
7501 Page Avenue  
St. Louis, Missouri 63133 ■ USA  
800-548-4873 ■ www.steris.com


**CH REP**  
**STERIS GmbH**  
c/o BDO AG  
Längfeldweg 116A  
CH-2504 Biel-Bienne  
Switzerland

**UK REP**  
**STERIS Solutions Limited**  
Chancery House Rayns Way  
Watermead Business Park  
Leicester LE7 1PF UK

**EC REP**  
**STERIS Ireland Limited**  
IDA Business and Technology Park  
Tullamore ■ County Offaly ■ R35 X865 ■ Ireland  
24 Hr. Emergency No. +44 (0) 1895 622639

**MDI**  
Imported to Canada by:  
**STERIS Canada Sales ULC** ■ 6-88B East Beaver Creek Road  
Richmond Hill, ON L4B 4W2 Canada ■ 800-661-3937

2D91-Q0M(B)(923) 63775

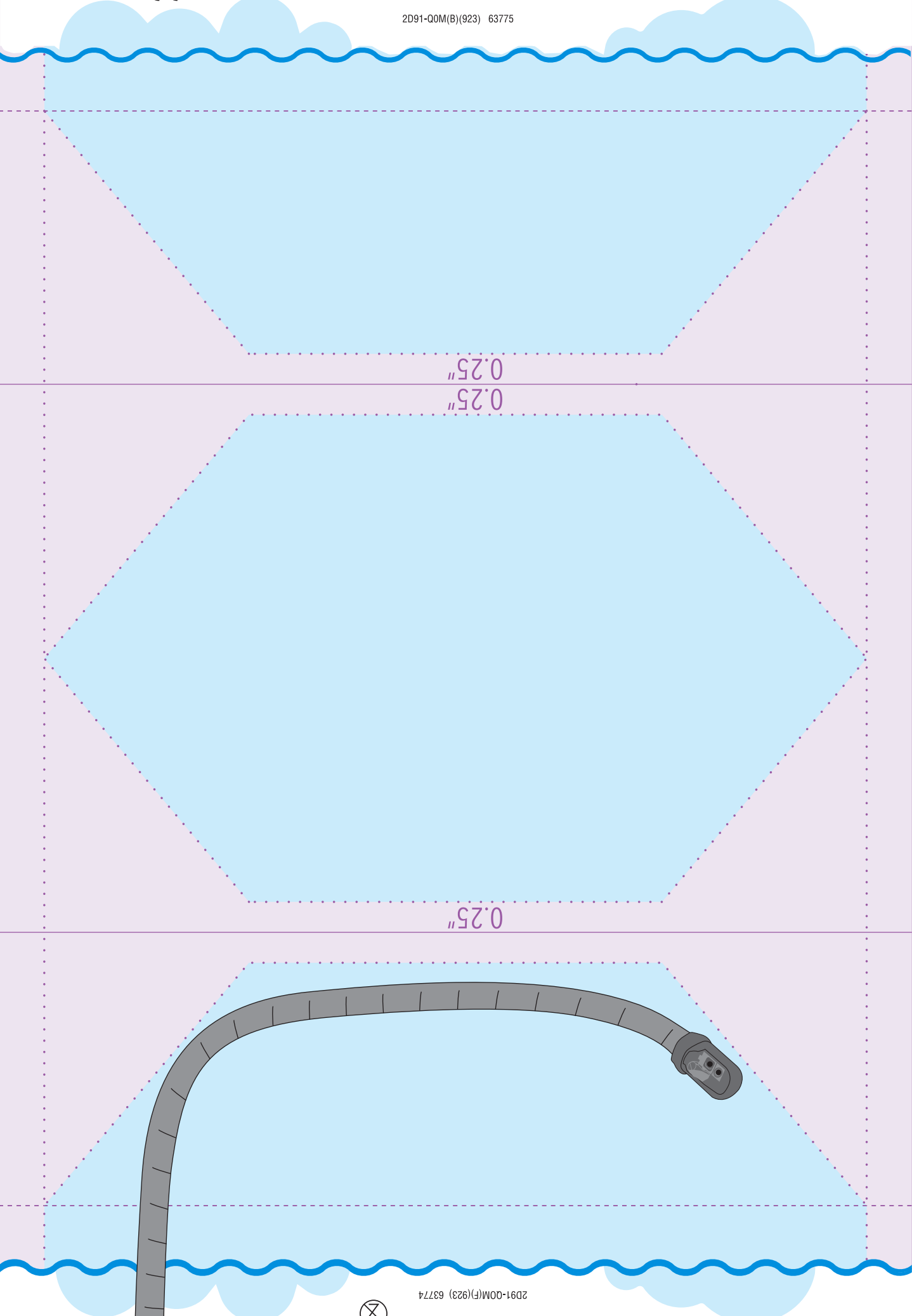

**REF 2D91QX**



**Discard packaging and sponge immediately after use FOR SINGLE USE ONLY**

⊗

2D91-Q0M(F)(923) 63774

**CONTENTS:** 1 Sponge with 215 ml (7.3 fl. Oz.) of Enzymatic Detergent

**For Surgical Instrument Application**

1. Carefully tear the pouch in an upward motion so the package opens.
2. While still in the pouch, lightly squeeze excess pre-diluted enzymatic detergent from the sponge.
3. Remove the sponge from the pouch and wipe down the exterior of the instrument.
4. Place the distal end of the endoscope into the pouch and suction the remaining enzymatic detergent through the biopsy suction channel according to scope manufacturer's guidelines.
5. Discard the sponge and used enzymatic detergent solution using approved methods for materials that have been in contact with possible infectious waste.

**For Endoscope Bedside Cleaning**

**INSTRUCTIONS FOR USE:**

- Review Original Equipment Manufacturer's guidelines regarding the care and cleaning of individual instruments.
- Use appropriate infection control principles and wear appropriate PPE.
- Review Original Equipment Manufacturer's guidelines regarding the care and cleaning of individual instruments.

Revital-Ox Bedside Complete Pre-Cleaning Kit is an all-in-one pre-diluted enzymatic detergent and impregnated sponge formulated to remove protein and lipid soils from the exterior and interior of flexible or rigid endoscopes and accessories or surgical instruments during point of use treatment.



**STERIS**

**Revital-Ox™**  
Bedside Complete Pre-Cleaning Kit

